



INSTRUCTIONS FOR USE

COLUMBIA BLOOD AGAR (CA)

PRODUCTS

AS-895

Columbia Blood Agar (CA)

4 plates / pkg

INTENDED PURPOSE

TruPRAS™ Anaerobic Culture Media is intended for the transport, preservation, or cultivation of a wide variety of microorganisms from specimens to aid in the isolation of bacteria for in vitro diagnostic and / or research / general laboratory purposes.

INTENDED USERS

Scientists, laboratory, and healthcare professionals trained in anaerobic microbiology techniques working in areas such as clinical, research, industrial, pharmaceutical and veterinary applications.

FORMULATION*

CA is an enriched, non-selective medium intended for the growth of a variety of microorganisms, including fastidious organisms from a wide range of specimen types. CA is also suitable for antibiotic susceptibility testing using differential disk methods and for performing spot biochemical tests. It is supplemented with vitamin K₁ and hemin to support the recovery of fastidious anaerobic bacteria. Defibrinated sheep blood is added to provide essential growth factors and to allow for the observation of hemolytic reactions, such as the characteristic double zone of β-hemolysis produced by *Clostridium perfringens*. This medium is prepared, dispensed, and packaged under oxygen-free conditions using TruPRAS™ Technology to prevent the formation of oxidized products prior to use. This product is supplied ready to use, with no pre-reduction step required.

| | | |
|-----------------------------|-------|----|
| Pancreatic digest of casein | 10.00 | g |
| Meat peptic digest | 5.00 | g |
| Yeast extract | 5.00 | g |
| Heart pancreatic digest | 3.00 | g |
| Corn starch | 1.00 | g |
| Sodium chloride | 5.00 | g |
| Agar | 13.50 | g |
| Hemin | 5.00 | mg |
| Vitamin K ₁ | 10.00 | mg |
| Defibrinated sheep blood | 45.50 | mL |
| DI Water | 1.00 | L |

*Approximate formula. Adjusted and/or supplemented as required to meet performance criteria.

Final pH: 7.3 ± 0.2 at 25°C

Final weight: 16.0 g ± 1.6 g



PRECAUTIONS

For *IN VITRO DIAGNOSTIC USE* only. Utilize approved biohazard precautions and aseptic technique when using this product. This product is for use by properly trained and qualified personnel only. Sterilize all biohazard waste prior to disposal. This product is manufactured as a single use device.

Report serious incidents that occur in direct relation to this product to tech@biolog.com. As necessary, report serious incidents to the regulatory authority in which the user is established.

This product may contain components of animal origin. All components of animal origin have been sourced from Bovine Spongiform Encephalopathy- (BSE-) free and Transmissible Spongiform Encephalopathy- (TSE-) free countries. Certified knowledge of the origin of animal derived components does not guarantee the absence of transmissible pathogenic agents. It is recommended that Universal Precautions be observed.

When working with anaerobic culture media, the potential for ergonomic hazards may exist due to repetitive motions, awkward postures, improper bench/chair heights or poor lighting. Although it is beyond the scope and provision of products by Anaerobe Systems, it should be recognized and mitigated by the end user in the laboratory environment.

STORAGE AND SHELF LIFE

Storage: Upon receipt, store at room temperature (15 – 25°C) in original package until used. Avoid overheating or freezing. Do not use media if there are signs of deterioration (shrinking, cracking, or discoloration due to oxidation of media) or contamination. The expiration date applies to the product in its original packaging and stored as directed. Do not use product past the expiration date shown on the label.

Shelf Life: 90 days from the date of manufacture.

PROCEDURE

Specimen Collection: Protect specimens for anaerobic culture from oxygen during collection, transportation, and processing. Consult appropriate references for detailed instructions concerning collection and transportation of anaerobes. The selection of specimens for culture is made by physicians or scientists collecting the sample.

Methods for Use: CA should be inoculated directly with a specimen or from a broth that has been inoculated from a specimen. Streak plates with inoculum to obtain isolated colonies and immediately place into an anaerobic atmosphere, incubating at 35 – 37°C for 18 – 48 hours. Extended periods of incubation may be required to recover slower growing anaerobes. Detailed instructions for processing anaerobic cultures can be found in the listed references. As packaged, this medium constitutes a qualitative, manual method.

MATERIALS REQUIRED BUT NOT PROVIDED

Standard microbiological supplies and equipment such as loops, saline blanks, slides, staining supplies, microscope, incinerator / autoclave, incubators, anaerobic chamber / anaerobic jars, disinfectant, other culture media, and serological / biochemical reagents.

INTERPRETATION OF RESULTS

CA supports the robust growth of a wide variety of microorganisms from diverse specimen types. In addition, this media should support typical pigment production by *Prevotella melaninogenica* and typical double zone of β -hemolysis around colonies of *Clostridium perfringens*.



LIMITATIONS

CA will not provide complete information for the identification of bacterial isolates. Additional test procedures and media are required for complete identification. In some cases, CA may be overgrown with swarming *Proteus* spp. or *Clostridium* spp. It is recommended that a media such as Laked Brucella Blood Agar with Kanamycin and Vancomycin (LKV, catalog #: AS-112) and/or Phenylethyl Alcohol Agar (PEA, catalog #: AS-113) also be inoculated from a specimen to prevent such overgrowth and thus provide isolated colonies. Consult reference materials for additional information.

QUALITY CONTROL

The following organisms are routinely used for quality control testing at Anaerobe Systems using the specifications outlined in the CLSI document M22-A3: Quality Control for Commercially Prepared Microbiological Culture Media.

| Organism Tested | ATCC® # | Results | Special Reaction |
|--|---------|----------|----------------------------|
| <i>Bacteroides fragilis</i> * | 25285 | Growth | |
| <i>Prevotella melaninogenica</i> * | 25845 | Growth | Pigment † (tan color) |
| <i>Fusobacterium necrophorum</i> | 25286 | Variable | |
| <i>Fusobacterium nucleatum</i> * | 25586 | Growth | |
| <i>Clostridium perfringens</i> * | 13124 | Growth | Double zone of β-hemolysis |
| <i>Peptostreptococcus anaerobius</i> * | 27337 | Growth | |
| <i>Staphylococcus aureus</i> | 25923 | Growth | |
| <i>Enterococcus faecalis</i> | 29212 | Growth | |
| <i>Escherichia coli</i> | 25922 | Growth | |
| <i>Proteus mirabilis</i> | 12453 | Growth | |
| <i>Cutibacterium acnes</i> | 6919 | Growth | |
| <i>Clostridioides difficile</i> | 9689 | Growth | |

* Organisms recommended by CLSI for quality control testing of anaerobic blood agars.

† Pigment production may require more than 48 hours of incubation

User Quality Control: The final determination to the extent and quantity of user laboratory quality control must be determined by the end user.

If the nutritive capacity of this medium is to be tested for performance, it is recommended that the following ATCC® organisms be evaluated for growth.

| Organism | ATCC® # | Expected Results | Special Reactions |
|--------------------------------------|---------|------------------|----------------------------|
| <i>Bacteroides fragilis</i> | 25285 | Growth | |
| <i>Prevotella melaninogenica</i> | 25845 | Growth | Pigment † |
| <i>Fusobacterium nucleatum</i> | 25586 | Growth | |
| <i>Clostridium perfringens</i> | 13124 | Growth | Double zone of β-hemolysis |
| <i>Peptostreptococcus anaerobius</i> | 27337 | Growth | |

† Pigment production may require more than 48 hours of incubation

Physical Appearance: CA should appear opaque burgundy red in color.

ATCC® is a registered trademark of American Type Culture Collection.



REFERENCES

1. CLSI. *Principles and Procedures for Detection of Anaerobes in Clinical Specimens; Approved Guideline*. CLSI document M56-A. Clinical and Laboratory Standards Institute; 2014
2. Leber AL, Burnham CA, eds. *Clinical Microbiology Procedures Handbook*. 5th ed. 4 vols. Washington, DC: ASM Press; 2023.
3. Carroll KC, Pfaller MA, eds. *Manual of Clinical Microbiology*. 13th ed. 4 vols. Hoboken, NJ: Wiley-Blackwell; 2023.
4. Jousimies-Somer HR, Sutter VL, eds. *Wadsworth-KTL Anaerobic Bacteriology Manual*. 6th ed. Belmont, CA: Star Publishing Company; 2002.
5. CLSI. *Quality Control for Commercially Prepared Microbiological Culture Media; Approved Standard- Third Edition*. CLSI document M22-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2004.
6. Ellner PD, Stoessel CJ, Drakeford E, Vasi FA. *New Culture Medium for Medical Bacteriology*. *AM J Clin Pathol*. 1966;45(4):502-504.
7. U.S. Department of Agriculture, Animal and Plant Health Inspection Service. *Animal Health Status of Regions*. Published March 12, 2025. <https://www.aphis.usda.gov/regionalization-evaluation-services/region-health-status>
8. European Commission. *Note for guidance on minimising the risk of transmitting animal spongiform encephalopathy agents via human and veterinary medicinal products (EMA/410/01 Rev. 3)*. Published March 5, 2011. <https://op.europa.eu/en/publication-detail/-/publication/3392e464-ba89-4ae4-955c-a07f617c8e06/language-en>

GLOSSARY OF SYMBOLS

| SYMBOL | TITLE | DESCRIPTION | STANDARD | REF# |
|--------|---|---|---|-------|
| | Catalog number | Indicates the manufacturer's catalog number so that the medical device can be identified. | ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied | 5.1.1 |
| | Lot number/ Batch code | Indicates the manufacturer's batch code so that the batch or lot can be identified. | ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied | 5.1.5 |
| | Use-by date | Indicates the date after which the medical device is not to be used. | ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied | 5.1.4 |
| | Authorized Representative | Indicates the Authorized Representative in the identified country or jurisdiction. | ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied | 5.1.2 |
| | Do not re-use/ Single use only | Indicates a medical device that is intended for one single use only. | ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied | 5.4.2 |
| | Consult instructions for use or consult electronic instructions for use | Indicates the need for the user to consult the instructions for use. | ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied | 5.4.3 |
| | Temperature limit | Indicates the temperature limits to which the medical device can be safely exposed. | ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied | 5.3.7 |



| | | | | |
|------------|------------------------------------|---|---|-------|
| IVD | In vitro diagnostic medical device | Indicates that a medical device is intended to be used as an in vitro diagnostic medical device | ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied | 5.5.1 |
| CE | CE Mark European Conformity | Designates that the product labeled is authorized for sale in European countries. | EU IVDR (EU) 2017/746 | |

AUTHORIZED REPRESENTATIVE INFORMATION

EC REP Casus Europe B.V.
 Lange Vlietstraat 2b
 3513 BK Utrecht
 The Netherlands

CH REP Casus Switzerland GmbH
 Hinterbergstrasse 49
 6312 Steinhausen
 Switzerland

REVISION 3

Additions: Intended Use, Intended Users, Animal Origin Statement, Ergonomics Precautions, Serious Incident Report Contact Information, Glossary of Symbols

Changes: Title change from Product Insert to Instructions for Use. Room temperature from 20 – 25°C to 15 – 25°C. References updated. Contact information.

Deletions: None